



PLIERS:

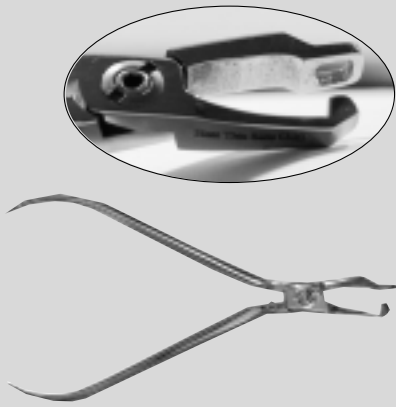

**Bite Plane Thermoplier
#82540**

TECHNIQUE: BITE PLANES

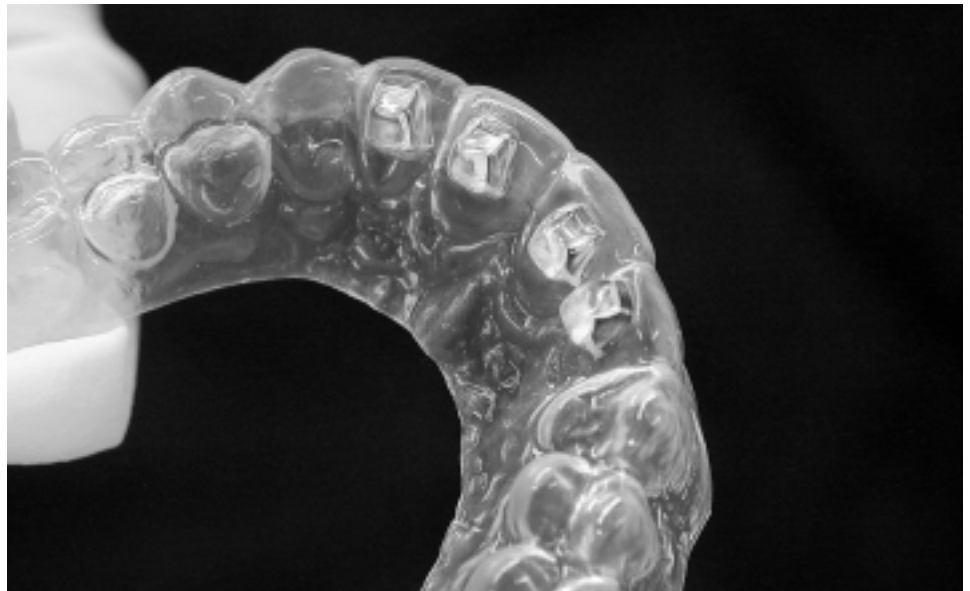

The Bite Plane Thermoplier is used to open the bite by placing bumps on the lingual surfaces of the upper anterior teeth.

1. Construct Essix appliance over working cast.
2. Seat appliance in patient's mouth.
3. Have patient bite down on articulating paper to mark where to place mumps on lingual surface.
4. Heat tip of pliers, check for correct temperature and squeeze pliers into marks.

EASY AS 1...2...3

Heating pliers with Essix plastics:

Essix A[®] = 175° • Essix C+[®] = 200°

1. HEAT TIP OF PLIERS



2. CHECK FOR CORRECT TEMP



3. SQUEEZE

