



## PLIERS:

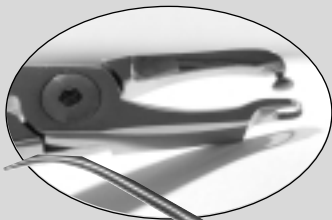


Hook Forming Plier #82550

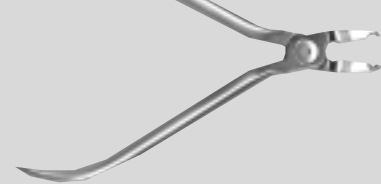


Slot Forming Plier #82570

## TECHNIQUE: CLOSING SPACE

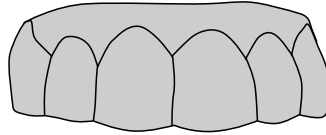


1. Construct full arch Essix<sup>®</sup> appliance.

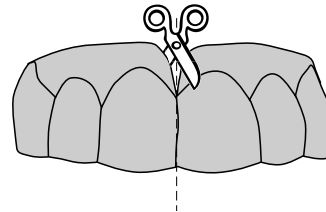

2. Cut appliance into two pieces at midline. Make sure and cut enough plastic away in order for teeth to move. If plastic is touching, it will impede movement.



3. Use Hook-forming & Slot-forming Thermopliers to create hook on labial of each incisor.

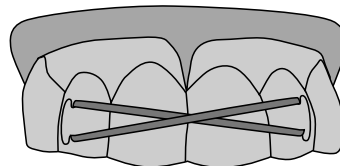

4. Seat appliance and attach 20g elastic in an X

## EASY AS 1...2...3

Heating pliers with Essix plastics:

Essix A<sup>®</sup> = 175° • Essix C+<sup>®</sup> = 200°

### 1. HEAT TIP OF PLIERS



### 2. CHECK FOR CORRECT TEMP



### 3. SQUEEZE



## Notes:

1. Movement is 1mm per month worn full-time except while eating.
2. Excellent procedure for closing reopened extraction space between cuspids.